


Six Steps to Connect HealthCap Application

1 Download **HealthCap** and register account.

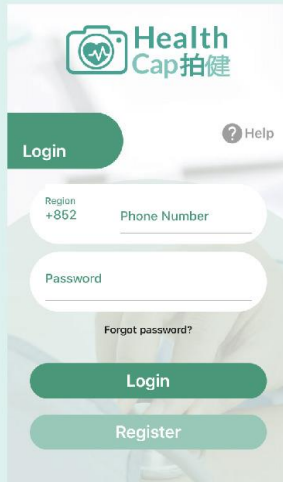

Scan the QR code to download the application



2 Turn on Bluetooth and Location Services (only for Android devices) on the mobile device.



Bluetooth



Location

3 Wear the cuff (pictured) and press the “設定” button on the Bluetooth blood pressure monitor when ready.




4 Open **HealthCap** on your mobile device and click on “HealthCap Bluetooth” located on the home page. The blood pressure monitor should then be connected to your mobile device automatically.



If "Not Connected" is displayed, please click "Reconnect".

5 Click "Start" to begin measuring blood pressure. Remember to stay relaxed.

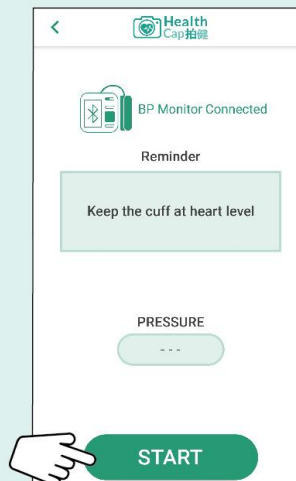

6 After the measurement, blood pressure records will automatically be stored in **HealthCap**. You can view your blood pressure report to better understand your conditions.

